



Horse Solarium Instructions



What's Included:

- 12 shatter-resistant infrared heat lamps
- Ample chain for over head hanging
- Long power lead with L5-30P plug

Solariums are fully PAT tested.

**Requires a 30-Amp breaker with a
NEMA L5-30R outlet receptacle.**



Hanging Instructions:

1. Unwrap all transport wrapping and ties.
2. Turn arm into parallel position on center rail and tighten wing nut.
3. Turn each arm end into position and tighten wing nuts.
4. Hang unit with chains provided to a safe overhead structure.
5. Screw lamps into brass holders.
6. Adjust unit to required height.
7. The recommended height is 24 inches (600mm) from the bottom of the outer lamp to horse withers.
8. Plug into an outlet and you are ready to go.

Usage Instructions:

- 10-15 minute sessions prior to work will help loosen up muscles.
- 20-25 minute sessions after work will relax muscles, decrease soreness, and aid in recovery.
- Use the solarium to dry a wet horse from turnouts or provide light exposure during winter months.
- Always monitor your horse for signs of discomfort.

To order replacement bulbs, visit
premierequestrian.com/light-bulbs/
or scan the QR code.



PREMIER EQUESTRIAN

PremierEquestrian.com 800-611-6109